

DrugWipe®

Drug Screening Solutions

Available through:



Alcohol Countermeasure Systems

60 International Boulevard
Toronto, Ontario M9W 6J2 CANADA

+1 416 619 3500

acs-corp.com
drugwipe.ca

ACS, ALCOHOL COUNTERMEASURE SYSTEMS and the "Molly" are trademarks of Alcohol Countermeasure Systems (International) Inc. and are used under license.

DrugWipe and WipeAlyser are registered trademarks of Securetec Detektions-Systeme AG and are used under license.



20181002

Simple to use, accurate in detection

Whether ensuring the safety of your employees, performing a road-side sobriety test, or testing in an emergency situation, the DrugWipe family of drug screeners is designed for results.

How does it work?

The test starts as soon as a saliva sample is given and the integrated ampoule is broken, releasing the liquid. The liquid helps the drug particles that are bound to the antibodies on the test strip migrate to the test line. A sequence of red lines appears on the read out window for each substance resulting in a positive test.

DrugWipe S Saliva Drug Screener

A more sanitary method of sample collecting compared to blood and urine tests. DrugWipe S requires the lowest sampling volume in the industry, making it ideal for the highest accuracy.



WipeAlyser® Portable Analysis Device (for DrugWipe S only)

WipeAlyser is easily controlled with the integrated touch display. Entries can be made with either a finger or the supplied stylus. You will be guided through the device operation step by step. Once the prepared test cartridge is inserted into the device, analysis and reliable results are provided in seconds and read electronically.



What drugs can these testers detect?

These drug screeners can detect a variety of drugs, including:

- Cannabis (THC/marijuana/hashish)
- Amphetamines
- Methamphetamines (MDMA/ecstasy)
- Cocaine (crack)
- Opiates (heroin/morphine/ketamine)

Why use a DrugWipe?

DrugWipe testers are simple to use and small enough to fit in your pocket. Stop waiting around for results—these testers take less than 10 minutes. DrugWipe testers are over 95% accurate, taking the guess work out of interpreting test results.

DrugWipe F Surface Drug Screener

Illegal practices, such as trafficking and handling, can leave trace amounts of drug residue on various surfaces. DrugWipe F permits those trace amounts of drugs to be tested from any surface.



DrugWipe A Universal Drug Screener

DrugWipe A test cassette has multi-purpose testing capabilities; sweat, saliva, surfaces—it can test them all! This screener is well suited for situations where error is not an option.